- en** Do not use if package is damaged
- de** Bei beschädigter Verp. verwerfen
- fr** Ne pas utiliser si l'emballage est endommagé
- es** No utilizar si el envase está dañado
- nl** Niet gebr. als verpakk. beschadigd is
- sv** Skall inte användas om förpackningen är skadad
- it** Non utilizzare se la confezione è danneggiata
- fi** Älä käyttää jos pakkaus vaurioitunut
- pt** Não usar se embalagem danificada
- da** Brug ikke, hvis pakken er beskadiget
- cs** Nepoužívejte, je-li obal poškozen
- no** Må ikke brukes hvis forpakningen er skadet
- lt** Nenaudoti, jei pakuotė pažeista
- lv** Nelietot, ja iepakojums ir bojāts
- et** Kahjustatud pakendi korral mitte kasutada
- ru** Не использ., если упак. поврежд.

- en** This product nor the packaging contain Natural Rubber Latex (NRL).
- de** Dieses Produkt sowie seine Verpackung ist latexfrei
- fr** Ni ce produit, ni l'emballage ne contiennent de Latex Naturel de Caoutchouc.
- es** Este producto o su empaquetado no contienen látex de caucho natural.
- nl** Dit product en zijn verpakking bevatten geen Natuurlijk Rubber Latex (NRL).
- sv** Varken denna produkt eller förpackning innehåller naturgummilatex.
- it** Né il prodotto né il confezionamento contengono lattice di gomma naturale.
- fi** Tämä tuote tai sen pakkaus eivät sisällä luonnonkumilateksia.
- pt** Este produto, tal como a sua embalagem, não contém látex de borracha natural.
- da** Hverken produktet eller emballagen indeholder naturgummilatex (NGL).
- cs** Tento výrobek ani obal neobsahuji přírodní gumový latex.
- no** Hverken produktet eller pakningen inneholder naturgummilateks.
- lt** Produktas sudėtyje nėra natūralios gumos latekso
- lv** Produkts nesatur dabīgo gumijas lateksu.
- et** Antud toode ega pakend ei sisalda naturaalseid lateksummi (NRL - Natural Rubber Latex).
- ru** Продукт и упаковка не содержат натурального латекса.

Mepilex® Lite with Safetac® Technology

Absorbent soft silicone dressing

Product description

Mepilex Lite consists of:

1. a Safetac wound contact layer
2. a thin, flexible absorbent pad of polyurethane foam.
3. an outer film which is vapour permeable and waterproof

Safetac Technology

Safetac is a patented soft silicone adhesive technology that minimises pain to patients and trauma to wounds. Safetac technology is less painful because it

1. tacks gently to dry surfaces, like skin, but not to moist surfaces such as open wounds
2. moulds to the skin's pores, covering more skin surface and spreading peel forces on removal to prevent skin stripping
3. seals the wound margins, ensuring exudate does not spread to the surrounding skin and minimising maceration

Mode of action

Mepilex Lite is thin and highly conformable, making it easy to keep the dressing in contact

with the wound surface and the surrounding skin, even in awkward or uneven areas. It absorbs exudate and maintains a moist wound environment.

As Mepilex Lite maintains a moist wound environment, supporting debridement, there might be an initial increase in the wound size. This is normal and to be expected.

Mepilex Lite can be cut to suit various wound shapes and locations.

Mepilex Lite can be used under compression bandaging.

Mepilex Lite can be used in combination with gels.

Intended use

Mepilex Lite is designed for the management of a wide range of non to low exuding wounds such as leg and foot ulcers, pressure ulcers, partial thickness burns, radiation skin reactions and Epidermolysis Bullosa. Mepilex Lite can also be used as a protection of compromised and/or fragile skin.

Instructions for use

1. Cleanse the wound in accordance with normal procedures.
2. Dry the surrounding skin thoroughly.
3. Remove the release films and apply the adherent side to the wound. Do not stretch.
4. For best result, Mepilex Lite should overlap the dry surrounding skin by at least 1-2 cm for the smaller sizes (sizes up to 12.5 x 12.5 cm) and 5 cm for the larger sizes in order to protect the surrounding skin from maceration and excoriation and fixate the dressing securely. If required, Mepilex Lite can be cut.
5. When necessary, fixate Mepilex Lite with a bandage or other fixation.

Frequency of change

Mepilex Lite may be left in place for several days depending on the condition of the wound and surrounding skin, or as indicated by accepted clinical practice. A change in dressing regimen may result in an initial increased level of exudate that temporarily may require an increased frequency of dressing changes.

Precautions

In case of signs of clinical infection, consult a health care professional for adequate infection treatment. Do not use Mepilex Lite together with oxidising agents such as hypochlorite solutions or hydrogen peroxide. If reused performance of the product may deteriorate, cross contamination may occur.

Mepilex Lite should be stored in dry conditions below 35°C (95°F).

Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilise.

Mepilex and Safetac are registered trademarks of Mölnlycke Health Care AB.

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AUSTRALIA Mölnlycke Health Care Pty. Ltd.
14 Aquatic Drive, Building 1 - Ground floor,
Frenchs Forest, NSW 2086
Tel: +61 2 9453 1144

AUSTRIA Mölnlycke Health Care GmbH,
EuropastraÙe, Wagenseilgasse 14, 1120 Vienna
Tel: +43 1 278 85 42
Customer Service Tel: 0800 292 874

BELGIUM Mölnlycke Health Care NV/SA,
Berchemstadionsstraat 72, B2,
B-2600 BERCHEM (Antwerpen)
Tel: +32 3 2868 950

CANADA Mölnlycke Health Care Inc.,
2010 Winston Park Drive, Suite 100,
Oakville, Ontario, L6H 5R7
Tel: +1 905 829 1502

CHINA Mölnlycke Health Care AB,
Room D8-D9, 20th Floor,
No. 1, Xi Zhi Men Wai Street
Xi Cheng District, Beijing, 100044 P.R. China
Tel: +86 10 5128 8571

CZECH REPUBLIC Mölnlycke Health Care s.r.o.
Hajkova 274/722, 130 00 Prague 3
Tel: +420 221 890 517 (Reception)
Tel: +420 221 890 511 (Customer Service)

DENMARK Mölnlycke Health Care ApS,
Gydevang 33, DK-3450 Allerød
Tel: +45 48 16 82 68
Customer Service Tel: +80 886 810

ESTONIA Mölnlycke Health Care OÜ,
Vabadusmuuseumitee 4-67, EE-13522 Tallin
Tel: +372 671 1520

FINLAND Mölnlycke Health Care Oy,
Pitäjänmäentie 14, 3 krs, FI-000380 Helsinki
Tel: +358 201 622 300
Customer Service Tel: 0800 113 442

FRANCE Mölnlycke Health Care SAS,
13 Allée du Château Blanc, CS 70205
59445 WASQUEHAL Cedex
Tel: +33 320 122 555
Customer Service Tel: 0800 910 292

GERMANY Mölnlycke Health Care GmbH
Postfach 3265,
DE-40682 Erkrath-Unterfeldhaus
Tel: +49 211 92 08 80
Customer Service Tel: 0800 186 21 80

HUNGARY Mölnlycke Health Care Kft
HU-1134 Budapest, Dévai u. 26-28. III. em.
Tel: +36 1 477 3080

INDIA Mölnlycke Health Care India Pvt Ltd
Universal Business Centre
Second Floor, Universal Trade Tower
Sector 49, Gurgaon Sohna Road
Gurgaon 122 018, Haryana
Tel: +91 124 469008

ITALY Mölnlycke Health Care s.r.l.,
Via Marsala 40/C, IT-21013 Gallarate (VA)
Tel: +39 0331 714 011

JAPAN Mölnlycke Health Care KK,
4F Conceria Nishi-Shinjuku Tower's West
6-20-7 Nishi-Shinjuku,
Shinjuku-Ku, Tokyo 160-0023
Tel: +81 3 6914 5004

KOREA Mölnlycke Health Care Korea Co., Ltd
4th Fl, Hwasung Bldg, 89-4
Ogum-dong, Songpa-gu, Seoul 138-130, Korea
Tel: +82 2 3789 1402

LATVIA Mölnlycke Health Care AB,
Dzelzavas 120 M, Riga LV-1021
Tel: +371 7040 740

LITHUANIA Mölnlycke Health Care AB,
J. Galdyziø g. 3, LT 08236 Vilnius
Tel: +370 5 2274 58 37

NETHERLANDS Mölnlycke Health Care B.V.
Postbus 3196, 4800 DD Breda
Tel: +31 76 521 9663

NORWAY Mölnlycke Health Care AS
Postboks 6229 Etterstad, NO-0603 Oslo
Tel: +47 22 70 63 70
Customer Service Tel: 800 161 78

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SOUTH AFRICA Mölnlycke Health Care,
Manta Medical
Manta Place, Turnberry Office Park
48 Grosvenor Road, Bryanston
P O Box 69962, Bryanston 2021
Tel: +27 11 516 1713

SPAIN Mölnlycke Health Care S.L.,
Avda. De Bruselas, 38 B - 1a Planta
Poligono Industrial Arroyo de la Vega
ES-28 08 Alcobendas [Madrid]
Tel: +34 91 484 13 20
Customer Service Tel: 900 963 232

SWEDEN Mölnlycke Health Care AB
Gamlestadsvägen 3C, Box 130 80
SE-402 52 GÖTEBORG
Tel: +46 31 722 30 00
Customer Service Tel: 020 798 264

SWITZERLAND Mölnlycke Health Care AG
Heimstrasse 18, Postfach
CH-8953 Dietikon 1
Tel: +41 44 744 54 00
Customer Service

UNITED ARAB EMIRATES
Mölnlycke Health Care
301 White Crown Building
Sheikh Zayed Road, P.O. Box 212565, Dubai
Tel: +971 4331 98 84

UNITED KINGDOM Mölnlycke Health Care Ltd
Arenson Centre, Arenson Way
Dunstable, Bedfordshire, LU5 5UL
Tel: +44 870 60 60 766
Customer Service, Private
Tel: 0800 917 4919
Customer Service, Public sector
Tel: 0800 917 4918

USA Mölnlycke Health Care LLC
5550 Peachtree Parkway
Suite 500, Norcross, GA 30092
Tel: +1-800-882-4582

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